HF 9524

HBS Evacuator

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HF 9524

Contents

| | | |
|-----------|---|-----------|
| 1 | Symbol descriptions | 20 |
| 2 | Introduction | 21 |
| 3 | Scope of delivery | 21 |
| 4 | Scope | 21 |
| 4.1 | Inteded Use | 21 |
| 4.2 | Contraindications | 21 |
| 5 | Structure and basic components of the system | 22 |
| 5.1 | Main menu | 23 |
| 5.2 | Selection of operation mode | 24 |
| 5.3 | Setting the time for the delayed motor stop | 24 |
| 5.4 | Selection of suction activation method | 25 |
| 5.5 | Additional Symbols on the HBS Evacuator | 25 |
| 5.6 | Filter life indication | 26 |
| 6 | Specifications | 27 |
| 7 | Accessories | 28 |
| 8 | Safety guidelines | 28 |
| 9 | Servicing and manufacturer address | 30 |
| 10 | Cleaning and disinfection | 30 |
| 11 | Maintenance and service | 31 |
| 11.1 | Repair | 31 |
| 11.2 | Replacement of fuses | 31 |
| 11.3 | Technical safety check | 31 |
| 11.4 | Transport | 31 |
| 11.5 | Environmental directives..... | 32 |
| | Warranty card | 33 |

1 Symbol descriptions

| Symbol | Definition |
|--|--|
|  | CE-labelling |
|  | Attention |
|  | Manufacturer |
|  | Reference code |
|  | Serial number |
|  | Medical device / FDA Prescription device |
|  | Medical device |
|  | Non sterile |
|  | Keep away from sunlight |
|  | Dry storage required |
|  | Temperature |
|  Hinweis auf eFU | (Electronic) instruction for use |
|  | No intervention on patients with cardiac pacemakers. |
|  | WEEE-labelling |
|  | Green dot |

2 Introduction

Our products are exclusively intended for professional use by appropriately trained and qualified personnel and may only be acquired by them.

The HBS Evacuator is designed to remove and filter smoke and aerosols from the surgical field generated during electrosurgical and laser procedures.

The system uses a vacuum motor to draw surgical smoke from the operation area through a vacuum tube and into the filter, where a series of filters process the smoke. The filter is disposable and encapsulated, which facilitates the installation and replacement of the filter. The filter is fully enclosed to protect healthcare personnel from potential contamination during its change.

The air filtration process comprises of four stages. The first stage is a pre-filter that captures and eliminates large particles. The second stage is a ULPA (Ultra Low Penetration Air) filter that captures particles and microorganisms from 0.1 to 0.2 microns at an efficiency of 99.999%. The third stage comprises activated carbon, and the fourth stage is a woven fibreglass filter media that is used to minimize the amount of fine activated carbon particles that escape the filter.

The HBS Evacuator has various advantages, including a touchscreen display.

You can select and monitor the program mode, motor power level, airflow level, activation method, and filter life indication on the LCD screen. You can activate the device by using the screen button, pneumatic pedal, or an automatic RF sensor.

3 Scope of delivery

| | |
|------------------|---|
| HF 9524 | HBS Evacuator |
| HF9524-00 | Smoke Evacuator with 3-Port Filter |
| HF9979-XX | Power cable |
| | Instruction for use |

4 Scope

The product is used in HF surgery. It may only be used by trained and qualified professionals.


4.1 Inteded Use

A device for extracting smoke and aerosols. The device is used during electrosurgical procedures to extract and filter smoke, thereby ensuring visibility.

4.2 Contraindications

No contraindications are known.


5 Structure and basic components of the system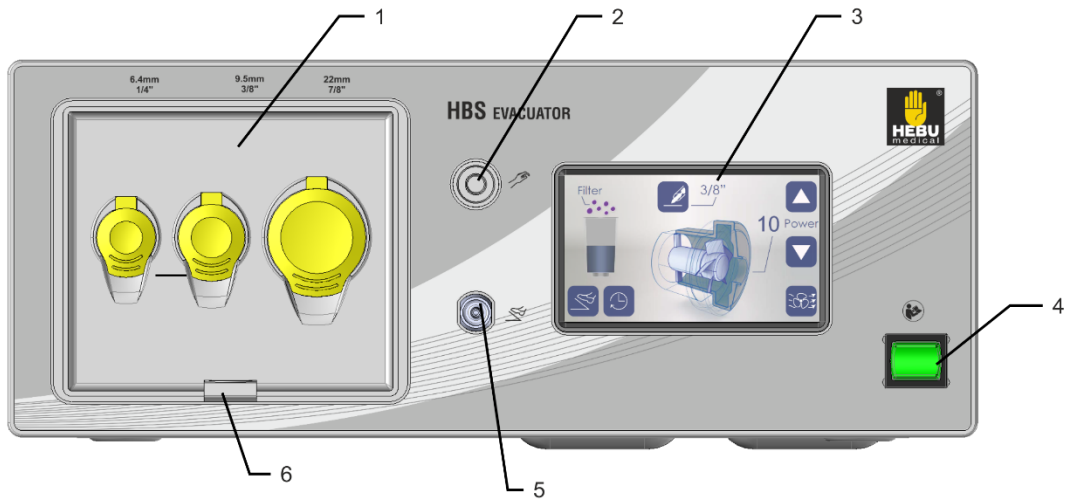

Front view of the HBS Evacuator

Located on the front of the unit are:

- (1) Filter
- (2) RF sensor input
- (3) HMI touch display
- (4) On/Off main switch
- (5) AIR footswitch input
- (6) Clamping for the filter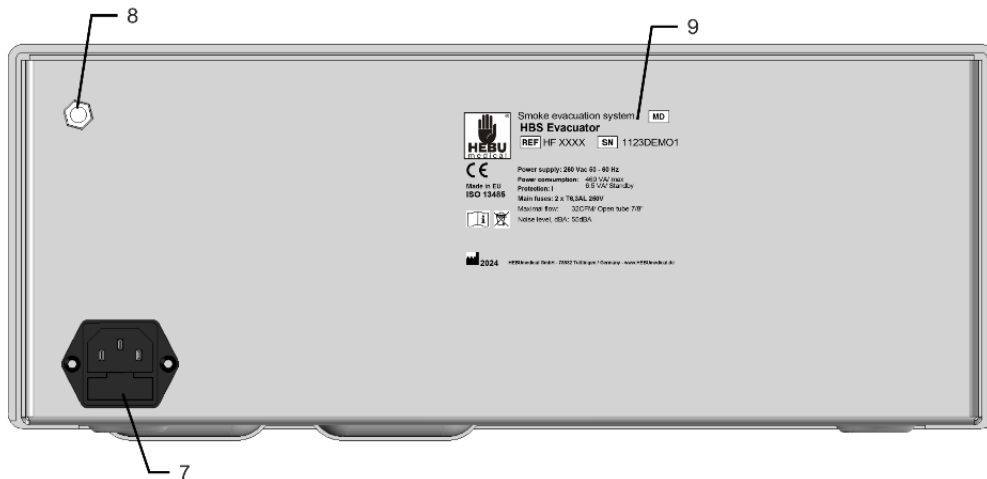


Rear view of the HBS Evacuator

Located on the back of the housing are:

- (7) Power input module with build-in fuses holder
- (8) Casing grounding
- (9) Label

5.1 Main menu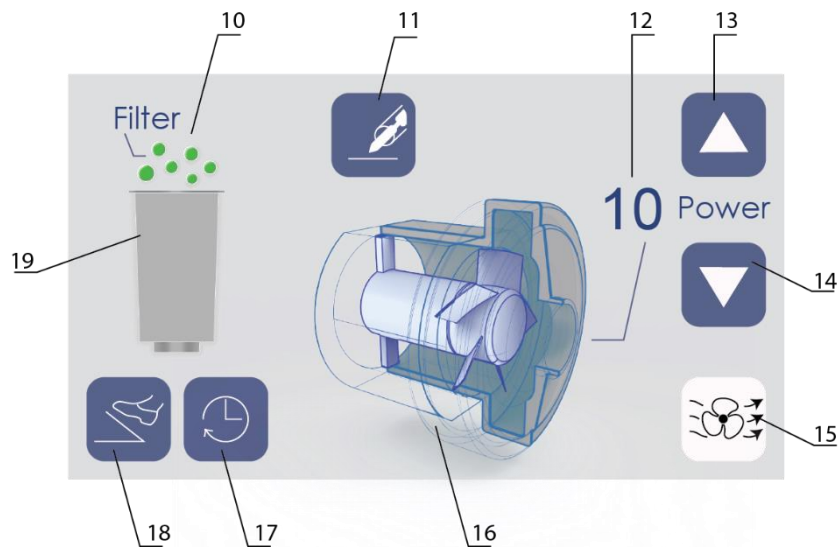


Main menu of HBS Evacuator

- **(10)** Filter life indication. See Chapter 3.1
- **(11)** Indication/Button for selecting the operating mode (See Chapter 5.2).
- **(12)** Indication of the selected power of the motor
- **(13)** Button to increase the power of the motor, in steps from 1 to 10, each step increases the power by 10%
- **(14)** Motor power reduction button, in steps from 10 to 1, each step reduces power by 10%
- **(15)** Suction enable/disable button. When the turbine is not running, the button is white
- **(16)** Indication of the state of the turbine - animation at the start of suction
- **(17)** Button for setting the delayed time for turning off the turbine (See Chapter 5.3)
- **(18)** Indication/Button for selecting the activation method
- **(19)** Indication of the level of airflow through the filter

5.2 Selection of operation mode

To access the "Operation Mode Selection" menu, simply press the button "11".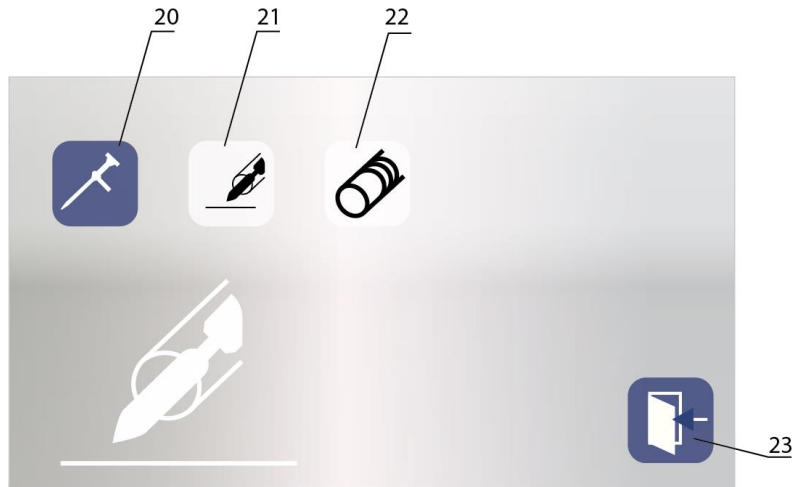


- **(20)** "Laparoscopic procedure" mode - if the system is connected to a surgical cannula during a laparoscopic operation
- **(21)** "Electrosurgical Mode" - if an electrocautery handle is used or an integrated smoke evacuation handle is used
- **(22)** "Open procedure" mode - an open procedure in which a smoke evacuation hose is used
- **(23)** Exit button - This button allows you to exit to the main menu and save the selection

5.3 Setting the time for the delayed motor stop


The system has a feature that enables users to set a time for turning off the motor after deactivating the RF energy. This allows the motor to continue running for a specific time to clear out any residual smoke from the operative area, after switching off by either the onscreen button, the footswitch or the RF sensor.

To access the settings menu, press button 17 from the main menu. This will allow you to set the time between 0 and 10 seconds.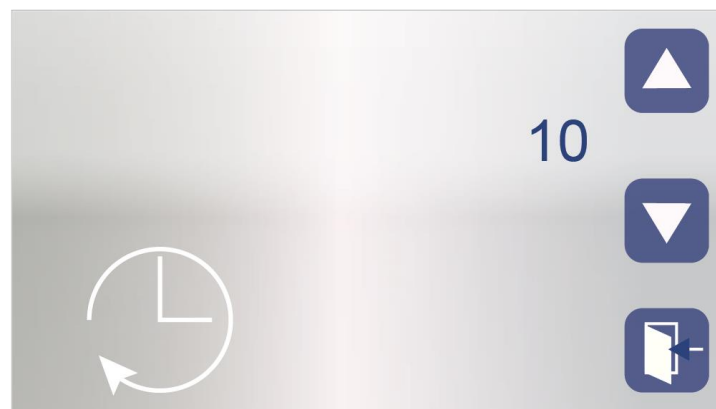

5.4 Selection of suction activation method

A pneumatic footswitch can be connected to socket 5 on the front panel. You can activate the motor using the pedal or button 15 on the main menu if pedal start is selected. If your system is equipped with an RF sensor, you can use button 18 from the main menu to access a menu that allows you to select the activation method and set the automatic mode. In automatic mode, the motor will start running when the RF sensor detects RF energy, and it will stop running when the sensor detects the power stop. It's important to note that if you select automatic start, the pedal and button 15 will not be able to control the motor.

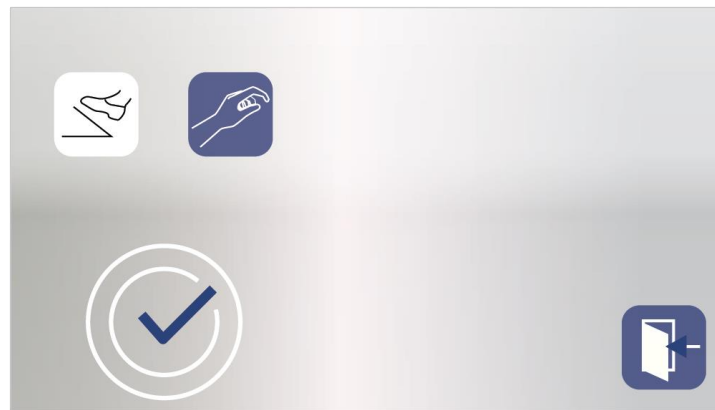The motor will be operated by pedal

The motor will be controlled by the RF sensor

5.5 Additional Symbols on the HBS Evacuator

| Symbol | Meaning |
|----------------------------------|------------------------------|
| Symbols on the frontplate | |
| | Consult instruction for use |
| | Footswitch plug-in connector |
| | RF sensor plug-in socket |
| Symbols on the backplate | |
| | Grounding |

5.6 Filter life indication

| Symbol | Definition |
|---|--|
|  | <p>New filter</p> |
|  | <p>The filter has been working for more than 16 hours.</p> |
|  | <p>The filter has been working for more than 33 hours.</p> |
|  | <p>The filter is not inserted or its life of 40h has expired</p> |

6 Specifications

| Maximum flow setting in "Open procedure" program | | |
|--|--|---------|
| 7/8" (22mm) | 35 CFM | 990 LPM |
| 3/8" (9,5mm) | 4.5 CFM | 130 LPM |
| 1/4" (6,4mm) | 2 CFM | 57 LPM |
| Device Dimensions (H x W x D) | 370mm x 144mm x 375mm | |
| Weight | 10kg | |
| Filter Dimensions (H x W x D) | 128mm x 112mm x 243mm | |
| Noise Level, dBA | 55dB | |
| Footswitch Activation | Standard pneumatic | |
| Remote Control Activation | RF sensor in range 350 - 600kHz | |
| Flow Control | Variable 1 - 10 Steps, in step of 1 | |
| OFF Time Delay Control | Setting from OFF to 10sec, in step of 1 sec | |
| Filter Status | | |
| Green | New | |
| Orange | Middle, >16h | |
| Red | High, >33h | |
| Violet | Service, 50h | |
| Vacuum level | Yes, bargraph | |
| Choice Of Three Programs | | |
| Open Tubing | 100% of Maximal flow | |
| Electrosurgical Pencil Attachment/Integrated Smoke Evacuation Pencil | 85% of Maximal flow | |
| Laparoscopic Tubing | 75% of Maximal flow | |
| Choice of activation method | Footswitch, Remote control | |
| Power supply | 110 – 260 Vac, 50 – 60 Hz | |
| Input power | Max 500VA (PF = 0.5) Standby 6VA (PF = 0.4) | |
| Fuses | 2 x T6.3A | |
| Construction | EN 60601-1 | |
| Protection class | I | |
| Grounding terminal | Yes | |
| Motor static suction | 21 kPa | |

7 Accessories

To avoid incompatibility and unsafe operation, use only the manufacturer's recommended accessories for working with the HBS Evacuator.

| | |
|---|--|
|  | <p>Follow the instructions to clean and prepare the accessories.</p> <p>Before each sterilization, check the connecting cables and handles for damage. Check for broken insulation, especially the electrode cables and the accessories used for endoscopic manipulations.</p> |
|---|--|

8 Safety guidelines

| | |
|---|---|
|  | <p>HEBU Medical mandates that the medical device should only be operated under the supervision of authorized or qualified personnel. The healthcare professional and surgeon are required to have adequate training and familiarity with the fundamental principles, application rules, and potential risks associated with HF surgery to ensure the safety of patients, personnel, and equipment.</p> |
|---|---|

This system is intended to be used exclusively for medical purposes. For safety reasons, the connection to the mains must be made through properly protected sockets, using the manufacturer's main connecting wires and plugs, or equivalent quality components in accordance with VDE 0107. To ensure safety, extension cords and splitters should not be used.

The device comes with a ground potential balance cable socket, which must be connected to an appropriate connection in the operating room to avoid any static electricity when the apparatus is repositioned.

| | |
|---|---|
|  | Repair of the appliance should be carried out only by HEBU Medical or by a representative authorized by HEBU Medical. |
|  | Appliance not to be used in areas with explosion risk. |
|  | The device, incl. connection cables to be checked for defects before each use. |
|  | Defects to be removed promptly. |
|  | Unplug the unit from the electrical outlet before checking the system components. |
|  | Make sure flammable materials used as cleaners, disinfectants, or tape removers have evaporated or been completely removed before starting the motor. |

| | |
|---|--|
|  | The use of non-flammable detergents for cleaning and disinfection is recommended. |
|  | Do not block either the pipe or the filter. If they become clogged or air movement through the filter is significantly restricted, the motor may overheat and cause damage to the unit. |
|  | The smoke evacuator creates a strong vacuum. Adjust the airflow and the position of the inlet end of the tube to prevent injury to the patient and to prevent aspiration of surgical materials and surgical specimens. |
|  | If the smoke evacuator is activated while the airflow is set to high speed, it may cause a sudden, strong suction. Check the airflow setting before starting work to prevent patient injury and prevent aspiration of surgical materials and surgical specimens. |
|  | To maximize patient safety, tubing or suction tips should not come into direct contact with tissue. Failure to do so may result in injury to the patient. |
|  | HBS Evacuator filters and disposable accessories are completely disposable. Please dispose of in accordance with local principles or regulations and hospital policy. |
|  | Only filters provided by the manufacturer have been proven to be compatible with the HBS Evacuator. Do not use other filters with this system. |
|  | Careful installation of hoses, adapters, and suction boxes is essential to prevent the motor from overheating and to avoid voiding the unit's warranty. |
|  | It is imperative that the installation of this equipment is done correctly to prevent any blockage of the intake and exhaust openings located at the bottom of the system. Failure to do so may result in reduced performance, damage and/or render the system inoperable, which will void the warranty. Please ensure that the installation is carried out in accordance with the manufacturer's guidelines to avoid any such consequences. |
|  | Attention: Please note that this device is NOT suitable for liquid aspiration. In order to avoid any damage to the device and to ensure optimal performance, it is important to install liquid collection devices if there is a possibility of liquid coming into contact with the filter. Failure to install a liquid collection device may result in filter clogging and electrical damage. |
|  | The filter should be changed according to the filter life guidelines. Filter life is determined by time only, not filter performance. |
|  | Changes or modifications not expressly approved by HEBU Medical could void the user's authority to operate the equipment. |
|  | If the products are used on patients with transmissible spongiform encephalopathy or HIV infection, we decline any responsibility for their reuse. |
|  | Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. |

9 Servicing and manufacturer address

Please contact your supplier in case of a problem that is insoluble for the operator or if the service is necessary. When installing the device, it informs you about the possibilities for technical maintenance and service at the specific point.

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

HEBUmedical GmbH
Badstraße 8
78532 Tuttlingen / Germany
Tel. +49 7461 94 71 - 0
Fax +49 7461 94 71 - 22
eMail: service@HEBUmedical.de
Web: www.HEBUmedical.de

10 Cleaning and disinfection

When Cleaning the device:
Before cleaning, disconnect the device from the mains.
Before connection the appliance to the mains, make sure that all flammable cleaning agents have completely evaporated.

Follow local hygiene guidelines.

- Clean all surfaces of the device (including the front) with alcohol-free cleaners and disinfectants.
- Wipe the LCD screen with a sponge or cloth soaked in clean water. Using aggressive cleaning agents will damage the display.
- Dry the device with a clean, lint-free cloth.
- Do not clean or disinfect surfaces of the case/front panel with flammable or explosive products. If their use is unavoidable, flammable or explosive products must be completely evaporated before switching on the device.
- The manufacturer recommends products with a pH between 9 and 10, e.g. MediClean forte by Dr. med. Weigert.
- Do not immerse the device in liquid.
- Do not sterilize the device with steam, ethyl oxide or other means.

11 Maintenance and service

11.1 Repair

Upon service: the repair of the device should be carried out only by HEBU medical or by an authorized representative.

Do not open the device housing. The HBS Argon does not contain any user-replaceable parts. The system may only be repaired by a qualified service engineer, who is specially trained to repair the HBS Argon.

Interventions of other persons can lead to cancellation of the guarantee and pose a risk for health.

11.2 Replacement of fuses

The fuses are located on the back of the device in a socket (7) on the rear panel, where the mains cable is also connected.

The fuses must be replaced as follows:

- Use a small screwdriver to remove the fuse holder.
- Replace defective fuses with the type described in technical data table.
- Put the fuse holder into its slot.

11.3 Technical safety check

To ensure proper and safe operation of the device, it is recommended to be inspected every 12 months by HEBUmedical or a HEBUmedical-approved service representative. At least every 12 months an inspection should be carried out as follows:

- Confirming the availability of instructions and user manual.
- Inspect the device and accessories for mechanical damage.
- Confirming the legibility of safety labels.
- Confirmation of secure and correct attachment of the unit.
- Confirmation that devices that are not functionally and operationally safe will no longer be used.

11.4 Transport

Prevent mechanical damage and moisture.

If the unit has been stored for a long period in a cold environment, wait before removing the protective packaging until the unit has reached room temperature.

During transportation, the standard safety measures additionally apply.

Only ever transport the unit in its original packaging.

11.5 Environmental directives

From the implementation date of the European directive 2002/96/EU as national legislation, the following regulations apply:

- Electrical and electronic units must not be disposed of with normal household waste.
- The consumer is required by law to return electrical and electronic devices at the end of their service life to designated public collection points or to the place of purchase.

Disposal details are set out by the applicable local legislation in the relevant country. The symbol on the product, the instructions for use or the packaging indicates the relevant applicable legislation. By sending end-of-life devices for reuse, material recycling or other forms of utilization, you are making a major contribution to environmental protection.

Please also send the unit packaging for environmentally responsible recycling after the end of the product's life.

Warranty card

On all electrical medical products we grant

We grant a warranty in accordance with the statutory and country-specific regulations on all electrical medical products (verification required in the form of an invoice or delivery note). The warranty period is two years. In the event of material or production errors, we provide free repair of the unit. Any attempt to carry out independent repairs will render the warranty cover null and void.

Damage caused by incorrect handling, overloading or normal wear and tear is not covered under the terms of the warranty.

In the event of a warranty claim and resulting repair, please enclose a copy of the invoice or delivery note (or batch number) with the instrument

Manufacturer and Aftersales Service Address:

HEBUmedical GmbH
Badstraße 8
78532 Tuttlingen
Tel. +49 7461 9471 -0
Fax +49 7461 9471 -22
service@HEBUmedical

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**Our vast range of products covers over 10 000 various instruments.
They can be found in Internet or request our catalog.**

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