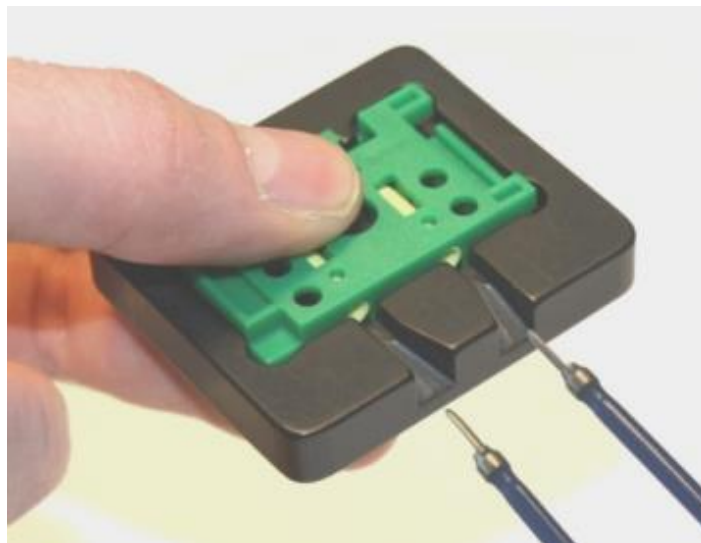
HEBU flexTIP

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1 General Information

This paper serves additional information and should be seen as an operation recommendation. In case damage to persons and/or material appeared out of improper use, any liability of HEBU medical GmbH is excluded.

1.1 Used Symbols

CAUTION!
Existing danger for person or material.

1.2 HEBU FlexTIP-System

All articles mentioned below are included in the HEBU flexTIP-System.

1.2.1 Forceps

HF 9600-11	HF 9630-16
HF 9600-16	HF 9630-19
HF 9600-19	

1.2.2 Forceps tips

HF 9601-01	HF 9601-02
HF 9602-01	HF 9602-02
HF 9603-01	HF 9603-02
HF 9604-01	HF 9604-02

1.2.3 Attach aid

HF 9600-01

1.2.4 Remove forceps

HF 9600-02

2 Scope

This instruction describes how to exchange forceps tips with the intention to decrease the risk of injury and to increase the saving of time. The instruction concerns only the HEBU flexTIP system.

The use of HF bipolar forceps takes place in HF surgery for the coagulation of tissue.

2.1 Inteded Use

HF bipolar forceps, flex Tip: A bipolar product covered with electrical insulation down to the working end. It is used in open surgery to coagulate vascular or tissue bundles for hemostasis. It is connected either to an electrosurgical diathermy unit or to an electrocautery generator that delivers the bipolar electrical current. It is typically embodied as a long, tweezer-like instrument with tapered, flat jaw members that are closed by the surgeon by applying pressure to the connected legs. It is made of metal and synthetic material in different sizes and shapes. They are reusable products.

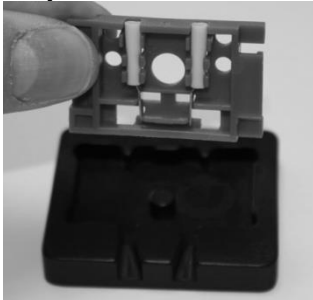
They are available with interchangeable tips. The flex-TIP system is available for replacing the forcep tips.

2.2 Contraindications

The HF bipolar forceps should not be used in conjunction with the central nervous or circulatory system.

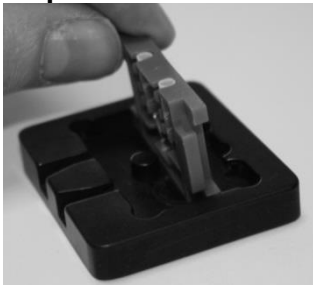
3 Attach of the forceps tips

step 1

- Pick up the carrier, so that the tips are pointing to the ground and the open side of the carrier is pointing to the front.

step 2

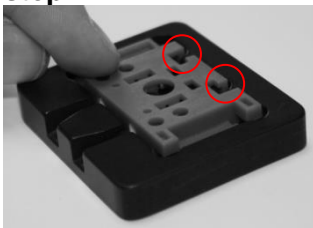
- Insert the carrier upright in the attach aid.

step 3

- Tilt the carrier to the front and push it backwards simultaneously.

step 4

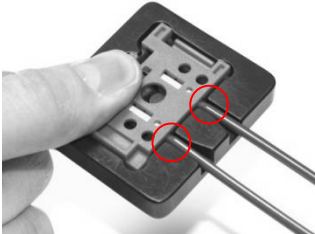
- The carrier is inserted correctly, when the connecting pieces are in the intended gap.

step 5

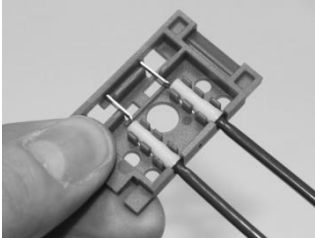
- Pick up the carrier with the attach aid in one hand. Push them against each other with thumb and forefinger.

step 6

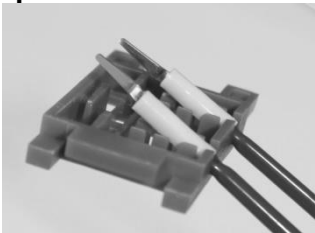
- Press with the other hand the HF forceps through the guiding joints into the openings of the tips until the metallic ends of the forceps completely disappear beneath the envelop of the tips

step 7

- Remove the forceps with the carrier out of the attach aid.

step 8

- Release the forceps by bending the carrier backwards.

4 Remove of the forceps tips

To remove the forceps tips grab one of the tips with the aid of the remove forceps and pull off the tip by performing rotating motion. Repeat this for removing the other tip.

Always pull off the tips away from body.

5 Cleaning, sterilization and maintenance

For information regarding cleaning, sterilization and maintenance please consult the respective manual instructions.

Since the forceps tips are disposables, the HEBU medical GmbH assumes no responsibility at reuse.

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HEBU medical GmbH non si assume responsabilità riguardo al riutilizzo delle punte, in quanto prodotto monouso.
HEBUmedical GmbH