HF 9527

HF 9528

HF 9529

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Contents

1	Used Symbols	10
2	Introduction	11
3	Scope	11
3.1	Intended Use	11
3.2	Contraindication	11
4	Handling	11
5	Construction	12
6	Safety Remarks.....	13
7	Preparation.....	13
7.1	Information on instrument preparation	13
7.2	Preparation at the place of use	13
8	Maintenance, inspection	13
8.1	Functional check	13
9	Storage.....	13
10	Warranty / Repair	14
11	Servicing and manufacturer address	14

1 Used Symbols

Symbol	Definition
	CE-labelling
	Attention
	Manufacturer
	Lot-description
	Reference code
	Medical device / FDA Prescription device
	Medical device
	Non sterile
	Keep away from sunlight
	Dry storage required
 Hinweis auf eIFU	(Electronic) instrucion for use

2 Introduction

Our products are exclusively intended for professional use by appropriately trained and qualified personnel and may only be acquired by them.

By purchasing this equipment trolley, you are now the owner of a high-quality product whose use and correct handling are described in the following. In order to minimize possible risks to patients and users, please observe these instructions carefully.

3 Scope

The use of equipment trolleys takes place in operating rooms.

Equipment trolleys are mainly used in operating rooms for the HF surgery. The equipment trolley was designed especially for HF devices. On the surface of the table there are special holder for a better hold of the device. The equipment trolley also has a deposition shelf compartment for instruments.

3.1 Intended Use

A trolley used as a means of transport to transport one or more electromechanical devices. This mobile trolley has one or more horizontal shelves in which the devices are positioned.

3.2 Contraindication

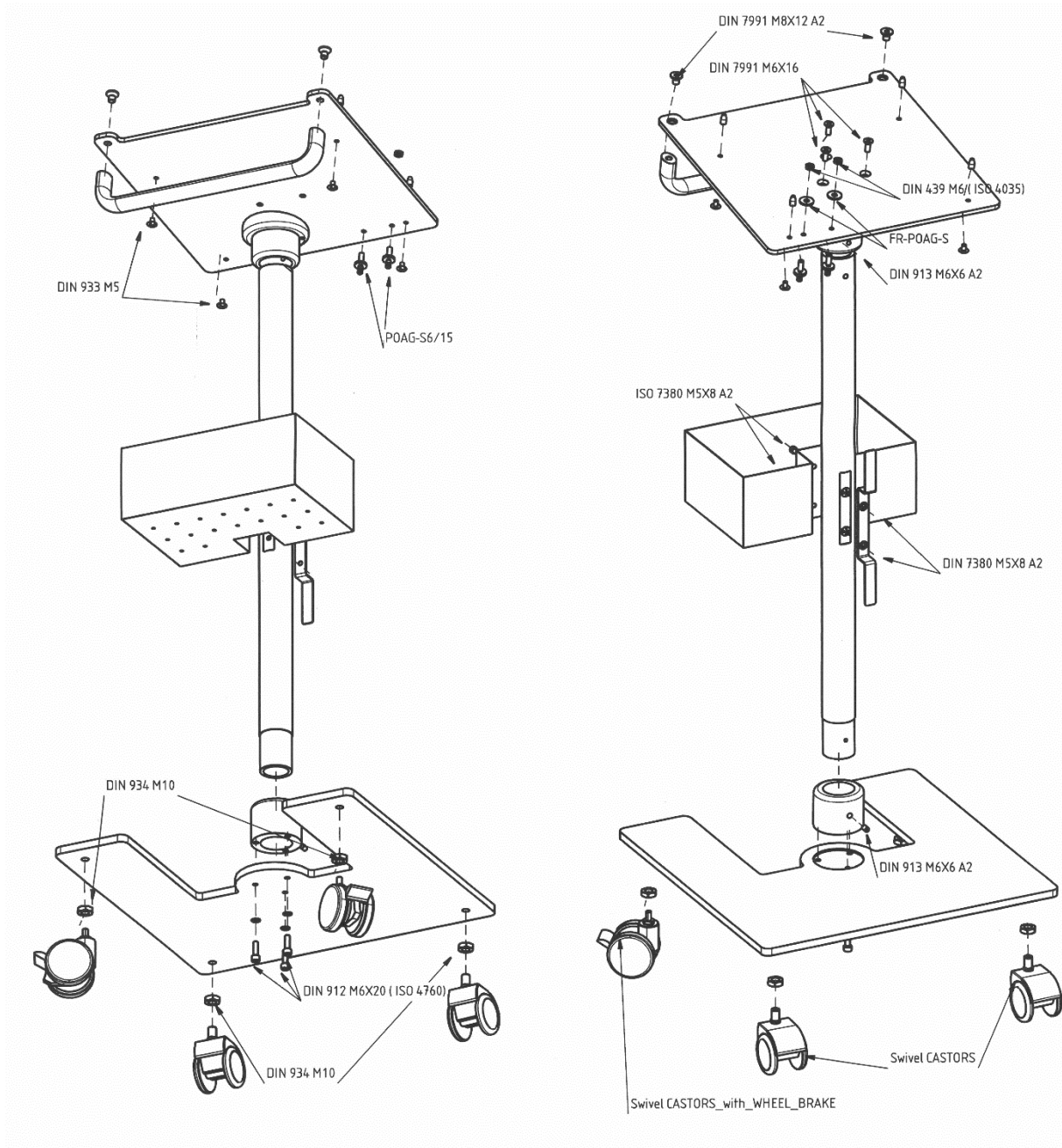
There is no known contraindication.

4 Handling

The product may only be used by appropriately trained and qualified personnel.

After each use, the equipment trolley must be checked for damaged. It is necessary to ensured that the HF device has a secure hold on the equipment trolley.

5 Construction

6 Safety Remarks

	The equipment trolley must be cleaned and disinfected before first use
	Please observe the additional information enclosed with the products
	Inspect the equipment trolley for damage after every period of use. A damaged trolley may not be used.
	Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

7 Preparation

7.1 Information on instrument preparation

- Use cleaning and/or disinfection agents with a pH-value within 9-10.
- Please observe manufacturer instructions regarding dosage, exposure time and renewal of solutions.
- Do not use hard brushes or coarse abrasive cleaners.

7.2 Preparation at the place of use

For cleaning the equipment trolley a wipe disinfection is recommended. We recommend using lint-free, soft cloths for manual cleaning. The cleaning agent should have no fixing effect.

8 Maintenance, inspection

Carry out the functional check mentioned above. Products with stains, which are bent, no longer function or which are otherwise damaged must be segregated!

8.1 Functional check

A newly purchased product must be subjected to a thorough visual and function check after its delivery and before each use.

Products must be checked for irregularities. Paying attention to cracks, fractures and the occurrence of corrosion.

Check instruments with joints for ease of movement. Carry out a function check in accordance with the intended application of the instrument.

Defective products must not be used and must have undergone the complete preparatory treatment process again before being returned.

9 Storage

Storage of the sterilized instruments in a dry, clean and dust-free environment at moderate temperatures between 5°C to 40°C.

10 Warranty / Repair

Our products are manufactured from high-grade materials and carefully checked prior to dispatch. However, even if used properly in accordance with their intended purpose they are subject to a greater or lesser degree of wear depending on their intensity of use.

This wear is technically induced and unavoidable.

Should faults occur independently of wear, please contact our customer services. Defective products should no longer be used.

They must undergo the complete preparatory treatment process before being returned.

11 Servicing and manufacturer address

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

HEBUmedical GmbH
Badstraße 8
78532 Tuttlingen / Germany
Tel. +49 7461 94 71 - 0
Fax +49 7461 94 71 - 22
eMail: service@HEBUmedical.de
Web: www.HEBUmedical.de

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**Our vast range of products covers over 10 000 various instruments.
They can be found in Internet or request our catalog.**

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