**HB 9150-12**  
**HB 9150-24**

# HEBU insta-hold

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DEUTSCH

**Operating Manual**  
Page 9

ENGLISH

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FRANÇAIS

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ESPAÑOL

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ITALIANO

**HB 9150-12**  
**HB 9150-24**

# HEBU insta-hold

## Inhalt

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# HEBU insta-hold

## Contents

|     |  |    |
|-----|--|----|
| 1   | Symbol descriptions .....                | 10 |
| 2   | Delivery programme .....                 | 10 |
| 3   | Scope .....                              | 11 |
| 3.1 | Intended Use .....                       | 11 |
| 3.2 | Contraindications .....                  | 11 |
| 4   | Operating .....                          | 11 |
| 5   | Cleaning and sterilisation .....         | 12 |
| 6   | Storage .....                            | 12 |
| 7   | Warranty .....                           | 12 |
| 8   | Environmental protection .....           | 12 |
| 9   | Manufacturer and servicing address ..... | 13 |

# 1 Symbol descriptions

| Symbol  | Definition                               |
|---|--|
|    | CE-labelling                             |
|    | Attention                                |
|    | Manufacturer                             |
|    | Lot-description                          |
|    | Reference code                           |
|    | Medical device / FDA Prescription device |
|    | Medical device                           |
|    | Non sterile                              |
|    | Keep away from sunlight                  |
|    | Dry storage required                     |
|  / <br>Hinweis auf eIFU | (Electronic) instruction for use         |

# 2 Delivery programme

|                    |   |
|--------------------|---|
| <b>HB 9150- 12</b> | 3 Pcs HEBU insta-hold /package incl. 5 fixation buttons<br>L*B: 480mm (19“) * 305mm (12“) |
| <b>HB 9150-24</b>  | 3 Pcs HEBU insta-hold /package incl. 5 fixation buttons<br>L*B: 480mm (19“) * 610mm (24“) |
| <b>HB 9150-99</b>  | Backup- Buttons, 5 pcs/package  |

### 3 Scope

Our products are exclusively intended for professional use by appropriately trained and qualified personnel and may only be acquired by them.

The HEBU Insta Hold is used in operations.

#### 3.1 Intended Use

Instrument Holder Mat: The HEBU Insta-Hold Mat is used to fix instruments during an operation. Almost all instruments in the operating environment can be fixed on any surface. It is a reusable product.

#### 3.2 Contraindications

No contraindications are known.

### 4 Operating

Congratulations: You are now the owner of the **HEBU insta-hold**, a precision engineered medical product from the company **HEBUmedical**.

This revolutionary development facilitates the work of any surgeon by providing a whole new standard of safety and convenience.

The **HEBU insta-hold** mat allows practically every type of instrument used in the operating environment to be firmly secured on different surfaces without the risk of magnetization (see Fig. 3).

The **HEBU insta-hold** mat can be cut to any optional shape (see Fig.1). If required you can fold a lateral pocket with fixation buttons (Fig.2) for a save positioning of the instruments (see Fig.3)

To use, place the sterile mat with the smooth side upwards on the patient next to the operating area

Fig. 1

Fig. 3

Fig. 2

The **HEBU insta-hold** fixing mat consists of silicone, avoid damage of sharp objects.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## 5 Cleaning and sterilisation

Please observe the following instructions:

- The fixing mat can be cleaned mechanically as well as manually. Always use a cleaning agent with pH value <10.
- The mechanical cleaning is possible with any alkaline agents.
- We recommend the use of soft brushes or cloths for the manual cleaning.
- Pay attention that the fixing mat is completely unrolled during the mechanical cleaning.
- Before first time sterilizing, peel off the protective film backing from the smooth side of the fixing mat! (Fig. 4)
- The fixing mat can be autoclaved at 134°C after an efficient cleaning
- For sterilization the cleaned mat can be convoluted

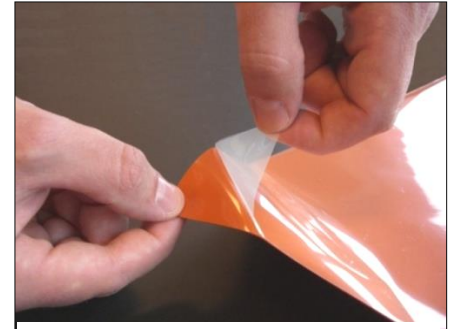
Fig. 4

If the products are used on patients with transmissible spongiform encephalopathy or HIV infection, we decline any responsibility for their reuse.

## 6 Storage

Store products in a dry, clean and dust-free environment at moderate temperatures from 5°C to 40°C. Protect from the effects of the sun's rays and artificial light.

## 7 Warranty

We provide a guarantee for **HEBU insta-hold** in accordance with statutory and country-specific legislation (proof of purchase by invoice or delivery note). The warranty period is two years. Damage caused by incorrect handling or normal wear and tear is not covered by the warranty. In case of warranty claims, please enclose a copy of the invoice or delivery note with the device.

## 8 Environmental protection

Any packaging materials and disused fixing mat must be sent for environmentally friendly recycling or disposal. For more information on this subject, consult your specialist dealer.

## 9 Manufacturer and servicing address

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

HEBUmedical GmbH  
Badstraße 8  
78532 Tuttlingen / Germany  
Tel. +49 7461 94 71 - 0  
Fax +49 7461 94 71 - 22  
eMail: [service@HEBUmedical.de](mailto:service@HEBUmedical.de)  
Web: [www.HEBUmedical.de](http://www.HEBUmedical.de)

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