# Stethoskop / Stethoscope

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Seite 3

**Operating Manual**

Page 9

DEUTSCH

ENGLISH

# Inhalt

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# Contents

1	Symbol descriptions .....	10
2	Introduction .....	10
3	Scope .....	11
3.1	Intended Use .....	11
3.2	Contraindications.....	11
4	Warning.....	11
5	Handling .....	12
6	Preparation and Sterilization.....	12
7	Maintenance, inspection .....	12
8	Packaging .....	12
9	Storage.....	12
10	Warranty / Repair .....	12
11	Manufacturer and servicing address.....	13

# 1 Symbol descriptions

Symbol	Definition
	CE-labelling
	Attention
	Manufacturer
	Lot-description
	Reference code
	Medical device / FDA Prescription device
	Medical device
	Non sterile
	Keep away from sunlight
	Dry storage required
	(Electronic) instruction for use

# 2 Introduction

Our products are exclusively intended for professional use by appropriately trained and qualified personnel and may only be acquired by them. By purchasing this instrument, you are now the owner of a high-quality product whose use and correct handling are described in the following. In order to minimize possible risks to patients and users, please observe these instructions carefully. Use, disinfection, cleaning and sterilization may only be performed by suitably trained specialist personnel.

### 3 Scope

Stethoscopes are mainly used for various auscultations and for blood pressure measurements.

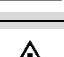
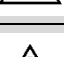
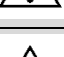
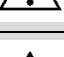
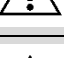
#### 3.1 Intended Use

A mechanical listening device for monitoring cardiac and pulmonary sounds. It usually consists of a membrane on the chestpiece, connected via a Y-fork with the earloop, which is placed in the ears of the user. Mechanical stethoscopes are usually manufactured in two variants: 1) a stethoscope for general clinical / hospital use; or 2) an anodized stethoscope for use by the cardiologist.

#### 3.2 Contraindications

No contraindications are known.

### 4 Warning

	Medical products are delivered in a non-sterile condition and must be cleaned, disinfected and sterilized prior to their initial use.
	The use of faulty instruments is in principle forbidden and they have to go through the whole cleaning process before return.
	Please take into consideration that through higher power a bigger damage of the tissue can result: f.e. on forceps: the power at the end of the jaw is higher than at the tip of the jaw
	Please observe the additional information enclosed with the products.
	Remove all protective sleeves and films prior to first using or preparation for use.
	The safe combination of different products or of products with implants must be reviewed prior to clinical application by the user.
	Avoid improper throwing or dropping of instruments
	Avoid mechanical overstressing of the instrument beyond the structural design, this can lead to breakage and deformation!
	A visual inspection of the instrument for damage and contamination must be carried out before each use!
	To prevent all contact corrosion, instruments with damaged surfaces must be separated immediately.
	If the products are used on patients with transmissible spongiform encephalopathy or HIV infection, we decline any responsibility for their reuse.
	Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## 5 Handling

The type of treatment must be determined in each individual case by the surgeon in cooperation with the internist and the anaesthetist.

For operational use in various surgical disciplines must be done by appropriately trained and qualified personnel.

## 6 Preparation and Sterilization

Stethoscopes are not suitable for machine reprocessing and sterilization. This can lead to irreparable damage. Therefore, cleaning must be done with a cloth and wipe disinfection afterwards. The user must observe the applicable national standards and guidelines. Thereafter, the disinfectants should be removed with a damp cloth. Due to the product design and the materials used, no limit can be set for maximum processing cycles.

## 7 Maintenance, inspection

After cooling to room temperature, the instruments must be visually inspected for protein residues and other contamination, paying particular attention to cavities, blocks, inclusions, pipes, and other inaccessible areas. Instruments which are not free of residues must be returned for a complete retreatment process.

Carry out the functional check mentioned above. Instruments with stains, which are blunt, bent, no longer function or which are otherwise damaged must be segregated!

## 8 Packaging

Compliant packaging of products for sterilization in line with ISO 11607. Packaging used must be suitable for the instruments and protect them from microbiological contamination during storage. The seal must not be under tension. HEBUmedical recommends container for sterilized packaging.

## 9 Storage

Store products in a dry, clean and dust-free environment at moderate temperatures from 5°C to 40°C. Protect from the effects of the sun's rays and artificial light.

## 10 Warranty / Repair

Our products are manufactured from high-grade materials and carefully checked prior to dispatch. However, even if used properly in accordance with their intended purpose they are subject to a greater or lesser degree of wear depending on their intensity of use.

This wear is technically induced and unavoidable.

Should faults occur independently of wear, please contact our customer services. Defective products should no longer be used.

They must undergo the complete preparatory treatment process before being returned.

## 11 Manufacturer and servicing address

Should you require the instructions for use in paper form, please use the contact details below. The instructions for use in paper form will be made available to you within seven calendar days of receipt of the request.

Alternatively, you can print out the electronic instructions for use yourself.

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**Our vast range of products covers over 10 000 various instruments.  
They can be found in Internet or request our catalog.**

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HEBUmedical GmbH