***HB9300***

# HUV20 Luft Reinhaltungssystem / HUV20 air purification system

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**DEUTSCH**

**Operating Manual**  
Page 10

**ENGLISH**

# Inhalt

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# Contents

1	Symbols .....	10
2	Introduction .....	10
3	Scope .....	10
3.1	Intended Use .....	10
4	Warning .....	10
5	Technical specifications .....	11
6	Construction .....	11
7	Operation .....	12
7.1	Application .....	13
8	Cleaning .....	13
9	Environmental conditions .....	14
10	Warranty / Repair .....	14
11	Manufacturer and servicing address .....	14

## 1 Symbols

Symbol	Definition
	Attention
	Manufacturer

## 2 Introduction

With the purchase of this product you receive a high-quality product, the proper handling and use of which is described below. In order to minimize risks and unnecessary stress for patients, users and third parties, we ask you to read the instructions for use carefully and to keep them safe.

The HEBU HUV20 is not a product according to "Medical Devices Regulatory - MDR" and is not subject to its restrictions.

## 3 Scope

The HUV20 Air Purification System is designed for use in closed residential areas such as business offices, shops, medical centers, etc.

### 3.1 Intended Use

The HUV20 Air Purification System is a mobile UV-C lamp for active disinfection against bacteria and viruses in the air. The system is intended for use in enclosed populated spaces, such as offices, shops or medical centres, etc. The HUV20 kills pathogens by direct exposure to UV light and simultaneously filters the air by means of forced convection.

## 4 Warning

### **DANGER:**

This is a UV risk group 3 product. This product emits high-intensity UV-light that can cause serious injuries to the skin and eyes. Avoid direct exposure of skin and eyes. Use this product only indoors with adequate protective measures for the staff.

Breaking/leaking of the UV-light emitting element has low impact on human health. In case of breaking / leaking, we recommend ventilating the space for at least 30 minutes. Collect with gloves any broken parts, contain them in a sealed plastic bag, and take them to the local recycling center. Do not use vacuum cleaner for the job.

## 5 Technical specifications


Power supply	220-240 Vac, 50-60 Hz
Input power	max 48 VA
Continuous mode of operation	Uninterrupted / programmable over the LCD display with 24 h timer and 2 individually adjustable On-Off cycles.
Safety option	Built-in motion sensor - with choice for activation
Battery life of the clock	Min 5 years
Fuse	2 pcs. T 1,0 A (slow)
Power cable	length 5m

UV-light emitter	UV-C - TUV 15W SLV/25
UV-C wavelength	254 nm
UV-C Radiation	10W / 100h
Emitter life cycle	9000 h
Forced convection flow	38 m <sup>3</sup> / h

Construction	EN 60601-1
Protection class	I
Earthing terminal	Yes
length x width x height	420 x 400 x 1100 mm
Weight	18 kg
Room size	4 x 4 x 2,3m (13 x 13 x 7,5 ft)

### UV-light source

Special power distribution: TUV 15W SLV/25 with light spectrum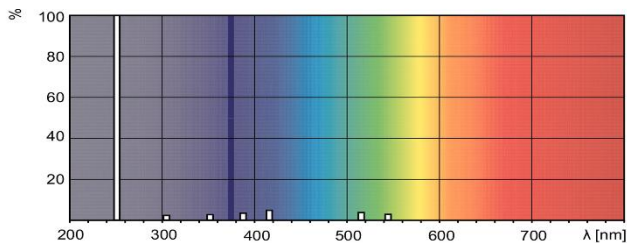


## 6 Construction

A robust frame out powder-coated of steel with individual parts out of aluminium, plastic and stainless steel.

## 7 Operation

Plug to mains and then use the main switch on the back of the lamp to turn on/off. UV-emission and convection are possible in both manual and automatic modes of operation. On initial power-up, the lamp is always in manual mode.


### Motion sensor activation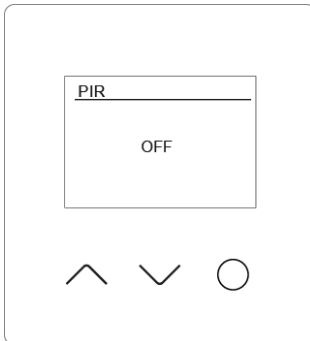

On initial power up, the motion sensor activation page is displayed. Switch the PIR sensor (**P**assive **I**nfra**R**ed sensor) on/off by pressing  $\wedge$  (ON) and  $\vee$  (OFF) buttons and final confirmation with the  $\bigcirc$  button.

If up to about 5 sec. no action is taken, the LCD timer switches to the last selected mode.

### Clock setting

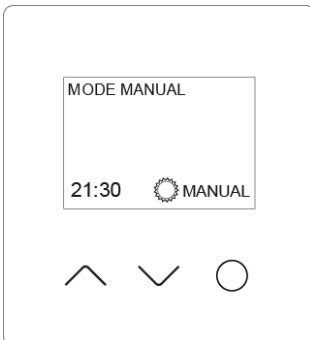
Clock setting is performed by a sequential selection of fields with the  $\bigcirc$  button and adjustment of values with  $\wedge$  or  $\vee$  buttons and final confirmation with the  $\bigcirc$  button.

The clock has a built-in battery and do not require re-setting on every power-up. Battery life is 5 - 10 years and qualified professional must perform replacement only.

### Manual mode

Turn the light on/off by pressing  $\wedge$  (ON) and  $\vee$  (OFF) buttons.

### Automatic mode

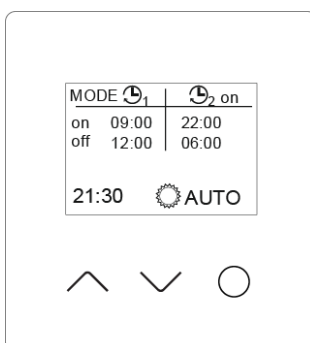In automatic mode, two separately programmable on/off cycles are available to the operator. Engaging automatic mode is done by multiple pressing the button  $\bigcirc$  until the MANUAL field is reached. Then, press  $\wedge$  to select AUTO and confirm with the button  $\bigcirc$ .

Programming the on/off cycles is done by sequential selection of the fields with the button  $\bigcirc$  selecting values with the buttons  $\wedge$  or  $\vee$  and final confirmation with the button  $\bigcirc$ .


The return to Manual mode is performed by successively passing through all the fields, by pressing the button  $\bigcirc$  until the AUTO field is reached. Then, press  $\vee$  to select MANUAL and confirm with the button  $\bigcirc$ .

☺ Turn on/off is always available by the main switch on the back of the lamp.

During the next power-up, the lamp initiates the last selected mode of operation – MANUAL or AUTO.


☺ Whenever the lamp is not operated in AUTO mode, it is recommended to unplug it from the mains.

## 7.1 Application
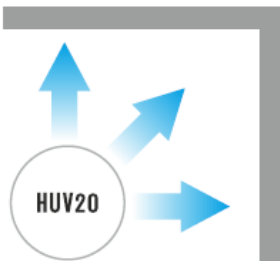
### **HUV20 shines in a corner or on a wall.**

For air purification, in particular to render aerosols harmless.

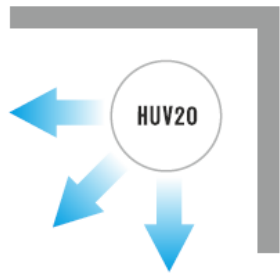### **HUV20 shines into the room.**

For surface disinfection and air purification. Operation only in empty rooms or in automatic switch-off mode via motion detector.

## 8 Cleaning

The air purification system can be cleaned with a cloth moistened with standard disinfectants and thus kept clean in the simplest way.

Notes:

- Perform cleaning only with the cable unplugged from mains power.
- Do not spray water on the UV-light emitters while lamp is ON.
- The lamp can be cleaned with any general cleaning/disinfecting means available in the medical practice.

## 9 Environmental conditions

### Storage

Ambient temperature	-10 to +60 OC
Relative humidity of air	10 to 85 %
Barometric pressure	500 to 1060 hPa

### Operation

Ambient temperature	+10 to +40 OC
Relative humidity of air	30 to 75 %
Barometric pressure	700 to 1060 hPa

## 10 Warranty / Repair

Our products are manufactured from high-grade materials and carefully checked prior to dispatch. However, even if used properly in accordance with their intended purpose they are subject to a greater or lesser degree of wear depending on their intensity of use.

This wear is technically induced and unavoidable.

Should faults occur independently of wear, please contact our customer services. Defective products should no longer be used.

They must undergo the complete preparatory treatment process before being returned.

## 11 Manufacturer and servicing address

HEBUmedical GmbH  
Badstraße 8  
78532 Tuttlingen / Germany  
Tel. +49 7461 94 71 - 0  
Fax +49 7461 94 71 - 22  
eMail: [service@HEBUmedical.de](mailto:service@HEBUmedical.de)  
Web: [www.HEBUmedical.de](http://www.HEBUmedical.de)

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**Our vast range of products covers over 10 000 various instruments.  
They can be found in Internet or request our catalog.**

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